


# Parts Manual

## Oxford® Ascend

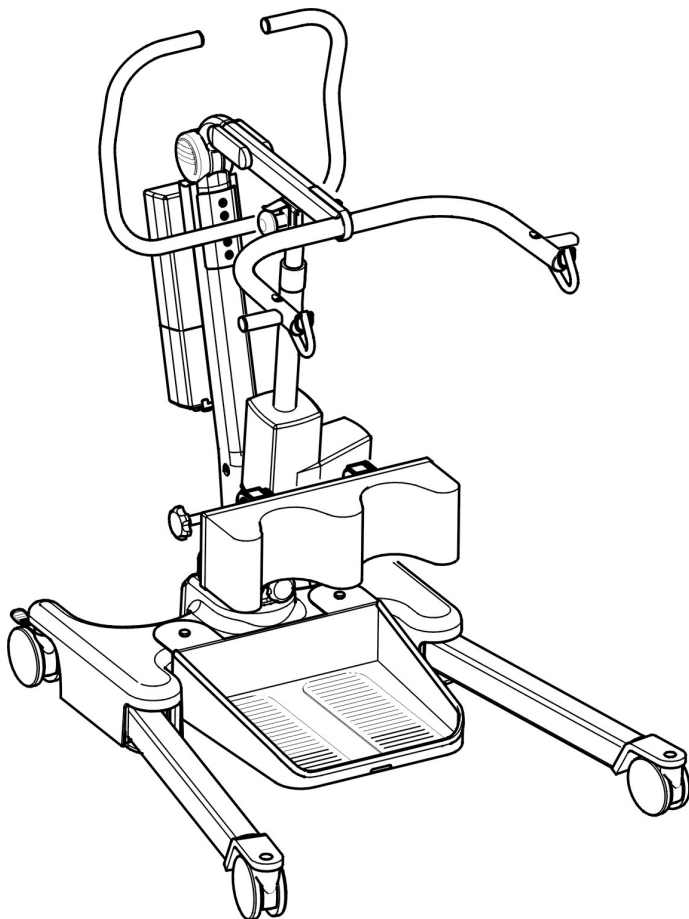

NO	PART NUMBER	DESCRIPTION	QTY
1	0Y0066	100mm rear castor (braked)	2
2	0Y0263	Straight-line steering device	1
3	0Y0073	100mm front castor inc. load bearing washer and tightening wrench (wrench not shown)	2
4	0Y0206	Mast extrusion including labels *	1
	0Y0308	Mast extrusion including labels **	1
5	0Y0207	Boom extrusion - Cow horn style (including boom end cap) *	1
	0Y0388	Boom extrusion - Cow horn style (including boom end cap) **	1
6	0Y0208	Push handle assembly (inc. bolts & fixings)	1
7	0Y0103	Casted boom to mast pivot joint (inc. connection bolts/fixings/spacers)	1
8	0Y0092	Base assembly (inc. labels, base casting and bottom plate, not internal parts)	1
9	0Y0209	Left leg extrusion (inc. fixings)	1
10	0Y0210	Right leg extrusion (inc. fixings)	1
11	39001370	Control box (Smart™ Monitor)	1
	0Y0211	Control box (Standard)	1
12	0Y0067	Battery pack	1
13	39001309	User handset (Smart™ Monitor only)	1
	0Y0075	User handset (Standard control box)	1
-	39001368	Calibration handset (Smart™ Monitor only) (not shown)	1
-	39001369	Actuator reset handset (Smart™ Monitor only) (not shown)	1
14	39001316	Smart™ Monitor retrofit kit (inc. control box, battery and user handset)	1
15	0Y0053	Charger and stand assembly (without charger lead)	1
16	0Y0071	Charger lead (UK)	1
17	0Y0072	Charger lead (EU)	1
18	0Y0212	Actuator (inc. location pins, sleeves, washers, nuts, labels) * <b>[OBSOLETE]</b>	1
	0Y0306	Actuator (inc. location pins, sleeves, washers, nuts, labels) **	1
19	0Y0069	Battery and controller bracket (inc. fixings)	1
20	0Y0093	Leg opening actuator - 35mm stroke (inc. bolts and fixings)	1
21	0Y0126	Pivot caps	2
22	0Y0102	Fixings kit (all bolts, screws and fixings)	1
23	0Y0077	Mast to base electrical connection kit (inc. wires)	1
24	0Y0078	Foot push pad assembly complete	2
25	0Y0112	Mast locking knob assembly (inc. plastic base insert)	1
26	0Y0632	Knee pad assembly (inc. all bolts, fixings and leg strap)	1
27	0Y0101	Foot plate assembly	1
28	0Y0129	Base to actuator electrical connection kit (inc. wires)	1
29	0Y0108	Boom to mast pivot plastic caps	2
30	0Y0076	Mast side caps	2
31	SA/2696	Sling retainer clips (Actuator/Boom)	2
-	0Y1061	Sling retainer clips (Cow Horn) - D ring (not shown)	10
-	0Y0687	Sling retainers - Oblong (not shown)	10
-	0Y0213	Decal set (no serial number labels) (not shown)	1
-	0Y0070	Handset clip (not shown)	2
-	0Y0128	Castor tightening wrench (not shown)	1

\* Last serial no. supplied: **0703L1146**. Please check with Customer Services prior to ordering.

\*\* Please check with Customer Services prior to ordering.

