


Sling Measuring Oxford Oxford Quickfit **Sling Measuring Guide:**

CAUTION: This document is intended as a guide only and in no way replaces an individual patient risk assessment. Patient specific conditions, functional independence measure (FIM) and weight distribution may influence the most suitable sling size as well as the most suitable type.



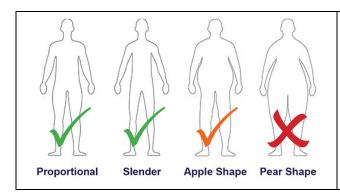

Body shape and size are important considerations when prescribing a sling. This quick reference tool is designed to help determine the compatibility of an individual with this type of sling.

= Good general compatibility.

= May be suitable.

x = Probably not compatible. **Use only with extreme caution**.

Key Dimension: Shoulder > Seat

Take the client measurement as shown in the diagram below and reference it against the corresponding measurements in the colour-coded chart to help determine the most suitable sling size.

R	Sling Size	Dimensional Range		Oxford slings have colour-coded
		Min	Max	binding matching this chart to help quickly identify the size.
\(\frac{1}{2}\)	XS	420mm	490mm	11/1/20
	S	470mm	540mm	
(<u>_</u>)	М	520mm	620mm	
	L	590mm	680mm	
	XL	650mm	740mm	

Selecting a sling for an individual involves more than simply measuring their size. Other considerations, as part of a comprehensive risk assessment, should include, but are not limited to:

- Type of transfer (e.g. bed to chair, chair to toilet, etc)
- · Level of support required
- · Patient conditions
- Physical and cognitive ability of the patient
- Abilities and limitations of the carer

- · Compatibility of the sling with other equipment
- Is a commode aperture required?
- Is the sling to be used for bathing?
- · Potential obstacles and hazards during transfer


It is critical to assess all possible risks when planning to lift and transfer a patient from one place to another.

Failure to assess the risk correctly may result in serious injury to the patient and/or caregiver.



Core Sling Dimensions: Oxford Quickfit

Quickfit						
Sling Size	Dimension A	Dimension B	Dimension C			
xs	640mm	470mm	960mm			
S	730mm	560mm	1120mm			
M	900mm	620mm	1210mm			
L	1110mm	700mm	1400mm			
XL	1390mm	910mm	1780mm			

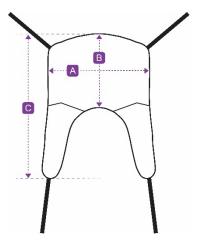

Custom Sling Service:

Joerns Healthcare offer a custom sling service should you have any non-standard or patient specific requirements. Please contact Joerns Healthcare for more information and guidance. Extended lead times apply.



^{*}All dimensions are subject to manufacturing tolerances.